



The Medical Power of Light

IntimaLase[®]

Laser vaginal tightening

- a solution for vaginal relaxation syndrome
- photothermal tightening of the vaginal canal
- minimally invasive, incisionless
- no anesthesia needed
- walk-in/walk-out procedure
- safe, quick and easy
- high success rate and patient satisfaction

Fotona
choose perfection

www.fotona.com

90589 CE ENG/6



What is IntimaLase®?

IntimaLase® is an innovative and unique, patented, non-invasive Er:YAG laser therapy for treatment of vaginal relaxation syndrome, a condition which is generally associated with overstretching of the vaginal canal during childbirth as well as with natural ageing.

How does IntimaLase® work?


The IntimaLase® treatment is based on a proprietary 2940 nm Er:YAG laser technology called Fotona SMOOTH®, which enables delivery of specially composed laser energy pulses to the vaginal mucosa tissue.

The precisely controlled Fotona SMOOTH® energy pulses are designed to induce thermal effects in the vaginal tissue, stimulating collagen remodeling and the synthesis of new collagen fibers. The end result of collagen neogenesis and remodeling is tightening of the vaginal canal.

Unique advantages of IntimaLase® for your patients

For the majority of patients, two sessions with a one-month interval are sufficient for significant tightening of the vaginal canal. No special pre-op preparation or post-op precautions are necessary. Patients can immediately return to their normal everyday activities.

After a single treatment, clinical results show a tightened vaginal canal, greater sexual satisfaction and significant improvement in a patient's quality of life.

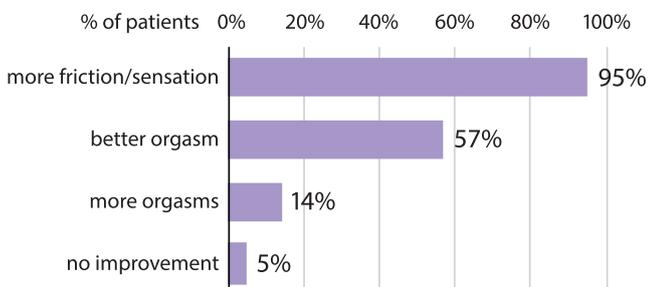

Patients' assessment of sexual gratification improvement after IntimaLase® treatment.

Gaviria J, Lanz J. Laser Vaginal Tightening (LVT) – evaluation of a novel noninvasive laser treatment for vaginal relaxation syndrome. LA&HA Journal of Laser and Health Academy, 2012(1); 46-58.

Excellent clinical results from numerous scientific studies

Since the introduction of Fotona SMOOTH® minimally invasive laser gynecology in 2012, more than 35 SCI (high Science Citation Index) publications have been published in the most highly respected, peer-reviewed international journals. These studies all show excellent results in the improvement of vaginal laxity and other genitourinary symptoms.



IntimaLase® treatment

Getting started with IntimaLase®

Visit the Fotona (www.fotona.com) or Laser and Health Academy (www.laserandhealth.com) websites to learn more about IntimaLase® and other Fotona SMOOTH® treatments, as well as training opportunities under the guidance of experts in medical laser technology.



"IntimaLase® has opened another window in my brain for helping my patients who were bothered by sexual dissatisfaction due to vaginal relaxation syndrome. Throughout the period of time I have used vaginal laser rejuvenation therapy, I have witnessed many patients repair the relationships with their partners after receiving the IntimaLase® procedure."

Dr. Jennifer Poning Lee



"95% of my patients assess vaginal tightness and sexual gratification as strongly or moderately improved after IntimaLase® treatment."

Dr. Jorge Gaviria